

AlcoHAWK® CA2010 Quick Guide

Be sure to read the entire owner's manual and follow procedures carefully before using this device.

Wait **20 minutes** after you smoke, eat or drink before testing. Failure to wait 20 minutes can give wrong BAC readings and damage the sensor.

Operating Instructions

Step 1) Insert a new mouthpiece each time you test another individual.

Step 2) **SLIDE UP AND HOLD** the ON/OFF power switch for 1 second until you hear the startup tones. The device will display "On" and count from 000 to 100% as it prepares the sensor for testing.

Note: The device will restart a count if the sensor is not ready within the first count.

Step 3) When you hear a **SINGLE BEEP** and the unit displays "READY - - -" take a **DEEP BREATH**, wrap your lips around the new mouthpiece and **BLOW STEADILY** (for at least 5 seconds) until a **DOUBLE BEEP** sounds to signal completion of the testing. Do not cover the **BREATH OUT** hole.

Note: If you do not blow within 30 seconds the device will display "Off" and power down. Continue from step 2.

Step 4) A circling line will display for approximately 5 seconds followed by "**RESULT**", your estimated BAC, and a long beep.

Note: The unit's backlight will turn from blue to red, display "**WARN**", and play alarm tones when the result is higher than 0.05% BAC.

Note: If you consumed very little alcohol, your actual concentration may be below 0.01% BAC and the unit will display 0.00% BAC and emit a long beep.

Note: After the result has been displayed for 20 seconds the device will display "Off" and power down.

Step 5) For more testing, wait two (2) minutes and repeat steps 1-4.



DO NOT DRINK ALCOHOL AND DRIVE

SEE OWNER'S MANUAL FOR HELP UNDERSTANDING THE RESULTS, PRECAUTIONS AND RE-CALIBRATION INSTRUCTIONS FOR RE-CALIBRATION THAT MUST BE ROUTINELY PERFORMED.

THIS DEVICE IS INTENDED TO MEASURE ALCOHOL IN THE HUMAN BREATH.

Calibration Log

Date First Used

Recalibration Date

Notes

Date First Used	Recalibration Date	Notes