





AlcoHAWK® PT500 Quick Guide

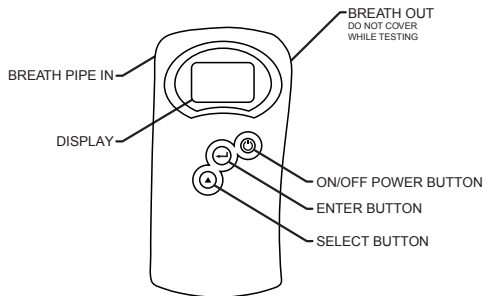

Be sure to read the entire owner's manual and follow procedures carefully to obtain optimal results.

Operating Instructions

Wait 20 minutes after the test subject smokes, eats or drinks before testing. Failure to wait 20 minutes can give inaccurate BAC readings and damage the sensor.

- Step 1)** Press and hold the ON/OFF power button  for 1 second until you hear a BEEP. The unit reads "Initializing..." as it prepares for use.
- Step 2)** The screen displays "Test" and "Options". To start a "Test", press the ENTER button . The unit displays "Blow Now". Take a deep breath, wrap your lips around the new mouthpiece and blow steadily for five (5) seconds. The unit beeps when the test subject starts blowing and beeps again after obtaining the sample. Do not cover the BREATH OUT hole.
- Step 3)** The unit displays "Processing" while analyzing the sample. After about three (3) seconds, the unit then displays the estimated BAC.
- Step 4)** The unit shows the test result for 90 seconds before beeping twice and shutting off.
- Step 5)** To turn off the unit, press and hold the ON/OFF power button  for 1 second.

To toggle through available menu options, press the SELECT  button. See manual for details on the menu options.

THIS DEVICE IS INTENDED TO MEASURE ALCOHOL IN THE HUMAN BREATH.

DO NOT DRINK AND DRIVE.

SEE OWNER'S MANUAL FOR RE-CALIBRATION INSTRUCTIONS THAT MUST BE ROUTINELY PERFORMED.