


Calibration Log

Date Used	Recalibration Date	Notes

AlcoHAWK® Slim Quick Guide

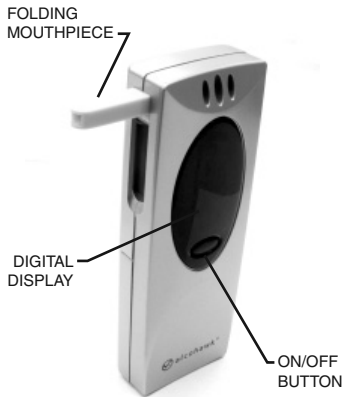

Be sure to read the entire owner's manual and follow procedures carefully to obtain optimal results.

Operating Instructions

WAIT 20 MINUTES AFTER SMOKING, EATING, OR DRINKING BEFORE USE. FAILURE TO DO SO CAN ALTER THE BAC READING AND CAN DAMAGE THE SENSOR.

- 1) Place a fitted cover over the mouthpiece each time you test another individual.
- 2) If it has been more than 24 hours since the device was last used, prime the sensor by blowing into the mouthpiece for 4 to 5 seconds before turning on the device.
- 3) **PRESS** the black **Power On/Off** button located on the front of the unit. A single beep will sound and the unit will count down from 60 as it prepares the sensor.

- 4) **WAIT** until the **RED LIGHT IS SOLID RED** and you hear a **SINGLE BEEP**.

Note: The device will not shut off until a test is performed OR until 15 seconds of non use elapses.

Note: The unit will restart a count down from 10 if the sensor is not ready within the first 60 second count down.

- 5) Take a **DEEP BREATH AND BLOW STEADILY** (for at least 5 seconds) until a **BEEP** sounds to signal completion of the testing.
- 6) **SEE OWNERS MANUAL FOR HELP UNDERSTANDING THE RESULTS, ERROR MESSAGES AND PRECAUTIONS.**
- 7) For more testing, wait one (1) minute and repeat steps 1 – 6.

THIS DEVICE IS INTENDED TO MEASURE ALCOHOL IN THE HUMAN BREATH.

DO NOT DRINK AND DRIVE.

SEE OWNER'S MANUAL FOR RE-CALIBRATION INSTRUCTIONS THAT MUST BE ROUTINELY PERFORMED.